**Quick Guide for
ViHealth**

Contents

Quick Guide.....	English	1-9
Kurzanleitung.....	Deutsch	10-18
Guida rapida.....	Italiano	19-27
Guía Rápida.....	Español	28-36
Guide rapide.....	Français	37-45

1 Connecting to the App

The ViHealth App allows you to transmit data after wearing the monitor, and supports real-time monitoring of your heart rate and ECG.

1.1 Downloading the App

App name: ViHealth Mobile

iOS: App Store

Android: Google Play

1.2 Installing the App

Install the App on an Apple product or Android-powered device, including smart phones and tablets.

Note: For Android devices, please accept the permissions requests that prompt when you run the App.

Compatibility

Devices that are compatible with the monitor and App meet the following minimum requirements:


iOS: iOS 9.0 or later

Android: Version 5.0 or later

Note: Android version 5.0, also known as Android Lollipop, was released in November 2014. Most devices manufactured after this date will likely have this version or a newer version installed.

1.3 Pairing the Monitor

1. Make sure that Bluetooth is enabled on your smart device and run the ViHealth App.
2. Follow the screen guide to sign in or sign up. You can also choose the guest mode.
3. Tap the detected monitor icon “VisualBeat” in the ViHealth App, then hold and touch the electrodes on the monitor to start pairing.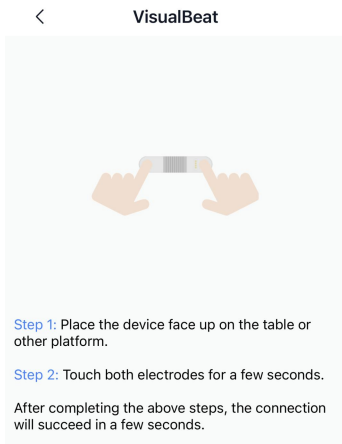

Caution: DO NOT pair the device in your smart device's settings.

1.4 Transmitting Heart Rate Data


The monitor will automatically transmit the latest heart rate data to the ViHealth App once paired.

2 Real-time Monitoring

When wearing the monitor, the ViHealth App allows you to monitor your heart rate and ECG in real-time on the Dashboard screen.

Your real-time heart rate and ECG are displayed on the Dashboard screen, but the heart rate data is still saved on the monitor and not in the App. Only ECG data can be saved on the Dashboard screen.

You can tap **10mm/mV** or **20mm/mV** to zoom in/out the ECG waveform.

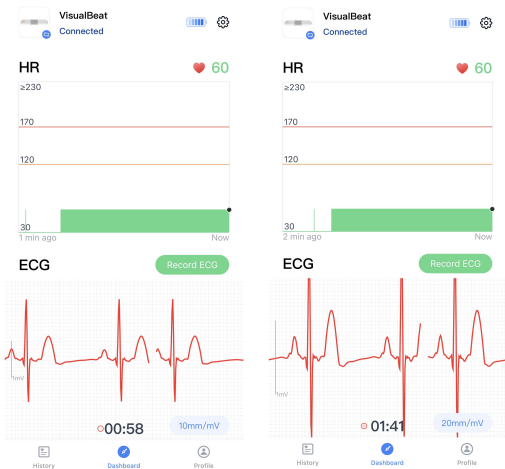Note: Keep your monitor connected to the ViHealth App when accessing real-time monitoring.

3 Recording ECG

To save the ECG data, you need to tap **[Record ECG]** to start recording on the Dashboard screen.

The maximum recording length is 30 minutes. If the recording time is less than 30 seconds, the recording will not be saved.

After successfully saving your ECG recording, you can review it on the History screen.


Note:

- In an ECG recording, only ECG data is recorded, not heart rate data.
- While recording ECG, remain calm and do not speak. Movements of any kind will falsify the measurements.

4 Reviewing History Data

You can review history measurement results on the History screen. Tap the desired recording to view detailed information. There are two different types of records:


- Heart rate recording, which is transmitted from the monitor.
- ECG recording, which is generated after real-time ECG recording.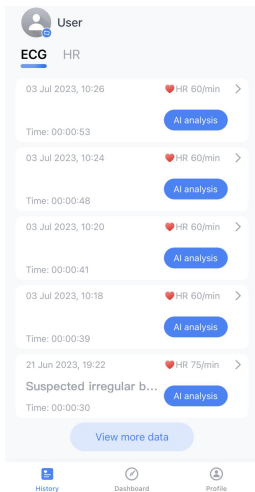

4.1 Heart Rate Recording

Detailed information is displayed in a heart rate recording, including measurement time, maximum heart rate, average heart rate, heart rate motion and trend curves.

The different colors in the heart rate zones signify setting heart rate targets. You can adjust the values of **[HR Target 1]** and **[HR Target 2]** in the Settings screen.

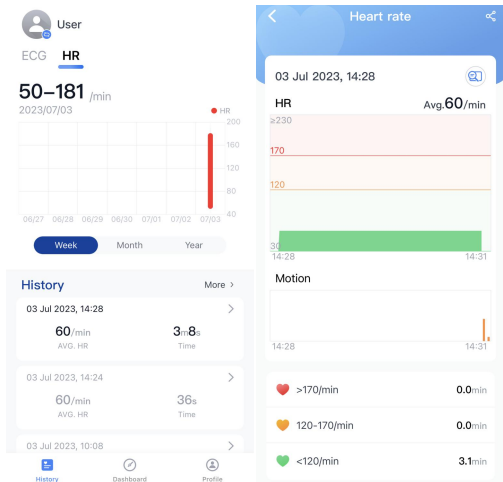Heart rate > [HR Target 2]

[HR Target 1] ≤ Heart rate ≤ [HR Target 2]


Heart rate < [HR Target 1]

Adding Notes

Slide down the screen to input notes of the recording.

Sharing Heart Rate Recording

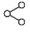Tap  to share the current heart rate recording as an image.


4.2 ECG Recording

The recording data will be displayed in rows of ten seconds. The maximum measurement time is 30 minutes.

Adding Note

Input notes of the recording.


Sharing ECG Record

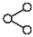
Tap  to share the current ECG record as a PDF file.

Displaying Full Screen Waveform

Tap  to view the ECG waveform in full screen.

Adjusting Waveform Amplitudes

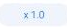
Tap  to adjust the amplitude for the ECG waveform. Options: 0.5mV/1mV/2mV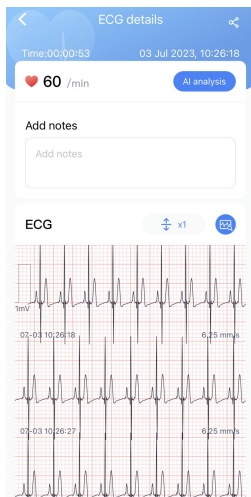

4.3 Deleting Records

To delete a record:

1. Swipe a recording to the left.
2. Choose "Delete" to delete the selected recording.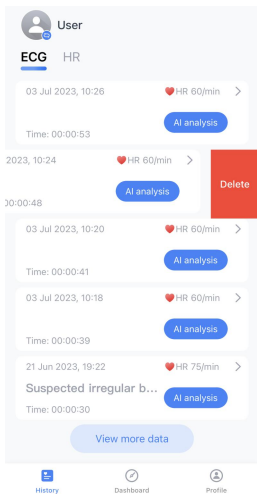


5 Setting Vibration Alerts

In the Profile screen, tap [**My device**] ->[**HR Target**]->[**Vibration**], then you can turn on/off heart rate vibration alerts.

When the detected heart rate is higher than [**HR Target 1**] value or [**HR Target 2**] value, the vibration alert will be automatically triggered. You can choose the desired target values on the [**HR Target**] screen.

Note:

- For [**HR Target 1**] alert, the vibration is triggered every 10 seconds.
- For [**HR Target 2**] alert, the vibration is triggered every 2 seconds.

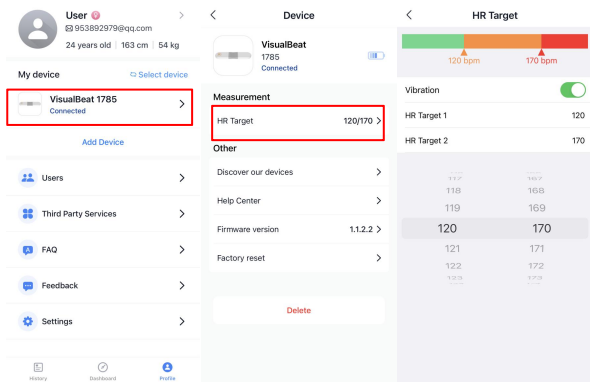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The screenshot shows the 'Cible HR' settings screen in the VisualBeat app. The screen is divided into three main sections: 'Dispositif', 'Mesure', and 'Cible HR'. The 'Dispositif' section shows the connected device 'VisualBeat 1785'. The 'Mesure' section shows the 'Cible HR' setting with a value of '120/170'. The 'Cible HR' section shows a bar chart with two target heart rate values: 120 bpm and 170 bpm. Below the bar chart, there is a 'Vibration' toggle switch which is turned on. Underneath, the 'Cible HR 1' is set to 120 and 'Cible HR 2' is set to 170. At the bottom, there is a table showing a range of heart rate values from 117 to 179 bpm.

117	167
118	168
119	169
120	170
121	171
122	172
123	173
124	174
125	175
126	176
127	177
128	178
129	179